


Ambu® King LTS-D™

Laryngeal Tube

Pediatric
sizes now
available

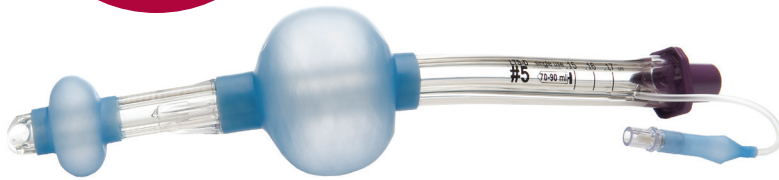

Key Benefits

- Seals in the esophagus and oropharynx to provide positive pressure ventilation
- Single inflation port
- Drain tube for gastric and suction catheters
- Phthalate and latex free material
- Color-coded full range of sizes

Ambu King LTS-D

The Ambu King LTS-D laryngeal tube is a disposable, simple to use alternative airway device that provides superior patient ventilation. The King LTS-D allows the passage of the gastric tube through a separate channel.

The King LTS-D is designed with a straightened, beveled distal tip that assists in directing the airway posterior to the larynx and into the upper esophagus. Due to this unique configuration, there is minimal risk of the device entering the trachea.

A single port inflates both proximal and distal cuffs. The proximal cuff stabilizes the King LTS-D and seals the oropharynx. The distal cuff blocks entry of the esophagus, reducing the possibility of gastric insufflation. Ventilatory seal pressures of 30cm H₂O or more are achievable with the King LTS-D. Multiple distal ventilatory openings and bilateral ventilation eyelets facilitate air flow.

Ordering specifications

Item no.	Size	Type
KLTS420	0	Ambu® King LTS-D™ with Suction Port, Size 0
KLTS421	1	Ambu® King LTS-D™ with Suction Port, Size 1
KLTS422	2	Ambu® King LTS-D™ with Suction Port, Size 2
KLTS4225	2 ½	Ambu® King LTS-D™ with Suction Port, Size 2.5
KLTS403	3	Ambu® King LTS-D™ with Suction Port, Size 3
KLTS404	4	Ambu® King LTS-D™ with Suction Port, Size 4
KLTS405	5	Ambu® King LTS-D™ with Suction Port, Size 5



King LTS-D sizes 3, 4, 5

Sizing Information

Tube Size	Pediatric				Adult		
	Size 0	Size 1	Size 2	Size 2.5	Size 3	Size 4	Size 5
Connector Color	Transparent	White	Green	Orange	Yellow	Red	Purple
Patient Criteria	<5 kg	5-12 kg	12-25 kg 90-115 cm	25-35 kg 105-130 cm	4-5 feet (122-155 cm)	5-6 feet (155-180 cm)	greater than 6 feet (>180 cm)
Recommended Cuff Volume	10 ml	20 ml	35 ml	40-45 ml	50-60 ml	70-80 ml	80-90 ml
Maximum Cuff Pressure	60 cm H ₂ O						
External Diameter of the Tube	9 mm	9 mm	14 mm	14 mm	17.6 mm	17.6 mm	17.6 mm
Bronchoscopy Via Ventilation Lumen	< 3.0 mm	< 3.0 mm	< 4.0 mm	< 4.0 mm	< 6.0 mm	< 6.0 mm	< 6.0 mm
Suction Catheter	10 Fr	10 Fr	16 Fr	16 Fr	18 Fr	18 Fr	18 Fr



King LTS-D sizes 0, 1, 2, 2.5

Ambu USA
 6230 Old Dobbin Lane
 Columbia, MD 21045
 Tel. 800 262 8462
 Fax 800 262 8673
 www.ambuUSA.com

CE₀₀₈₆ US: Rx only